



Swedish Hospital Supply AB

UNI-BAG Datasheet

| Issuer: | Date: | Edition: | BASIC UDI-DI: | Designation: |
|----------------------------------|--------------------------------|-------------------------|------------------------|---------------------|
| SHS, Swedish Hospital Supply, AB | 2022-04-07. | 2. | B-17350026250188 | UNI-BAG |
| Production: | Responsible production: | Quality Manager: | Product holder: | Supplier: |
| Fristad Plast AB (fp) | Nicklas Forsström. | Nathaniel Forsström. | SHS AB | SHS AB |

Product Description

Urine collection bag with mounted funnel (UNISEX).

Intended Use

The urine collection bag, UNI-BAG is an exceptionally good alternative to other urine collecting aids in e.g. hospital wards, home care and elderly care. Uni-BAG comes with a mounted UNISEX funnel and is used in most situations and positions (lying down, sitting, standing).

Features and benefits

- **UNI-BAG** is a closed-environment urine collection bag that can replace other collection vessels and is suitable for both women and men in most situations (lying down, sitting, standing).
- **The UNI-BAG** comes with a mounted funnel and carrying handle that makes it easy to grip and hang up. UNI-BAG can be used several times a day thanks to a unique check valve.
- **UNI-BAG** makes it easier for staff. The use of UNI-BAG means that staff have no direct contact with urine and odors are retained in the bag.
- **UNI-BAG** saves time for staff and money because in most situations it can be used several times in a day. Maximum volume approximately 1 500 ml or use max 24 h.
- **UNI-BAG** is made in Sweden from the material Polyethylene.



Swedish Hospital Supply AB

Bag

Material: Polyethylene foil, LDPE, transparent, green print

Length: Outer edge Weld. 320mm + 5mm

Width: Outer edge Weld. 183mm + 5mm

Thickness: 0.10mm + 0.01mm

Print: According to the picture or attached sample.

Printing color: Green

Check valve

Material: Speedflex EMS No. 3-05. According to the attached sample.
Punched in – other foil printed part – of the bag that interacts with injection-moulded funnel receiver, welded to the bag.

Funnel receiver

Material: Injection moulded polyethylene see picture or attached sample.

Funnel

Material: Injection moulded polyethylene see attached sample.

Handle direction: Recommended, straight up. Adjustable for patient fit.

| Article No | UDI-DI-Cardboard | Pcs/carton | Weight/carton |
|------------|------------------|-----------------|----------------|
| 100002-A | 17350026250003 | 50 pcs/carton | approx. 2.5 kg |
| | 17350026250140 | 2000 pcs/Pallet | approx. 120 kg |

Packaging

Material: Unbleached corrugated cardboard box with inner bag made of transparent polyethylene.

Outer dimensions long: 400 mm

External dimensions width: 300 mm

External dimensions raised: 400 mm

Number of details/carton: 50 pcs.

Number of cartons/pallet layer: 8 pcs.

Number of layers/pallets: 5 pcs.

Tot. Number of carton/pallet: 40 pcs.



Swedish Hospital Supply AB

Carton labeling: UDI-DI of the respective packaging volume, LOT, date of manufacture (year, month, day), expiration date (year, month, day).

Production

Procedures: The product is produced in accordance with ISO 9001.

Environment

Procedures: The product is produced in accordance with ISO 14001.

Country of origin

Sweden

Sterile product

No

Shelf life

4 years, unbroken packaging

Inventory and storage

Store dry at room temperature.
Avoid exposure to high temperatures above 55 C

Waste management

According to local regulations. Plastic recycling or incineration.

Other information

HS Code/Commodity code: 39269097

<https://swehospitalsupply.se/en/>



Swedish Hospital Supply AB



Secure-Innovative-Sympathetic